


OPENING GUIDE



• Flip over and open.

INSERTION:

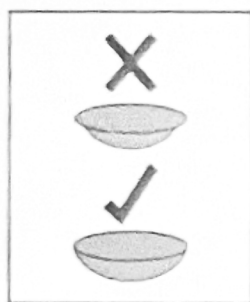

• Ensure lens is not inside out.




• Pull lower and upper eyelid.
• Place the lens gently on cornea.

LENS REMOVAL:



• Hold and pull lower eyelid while looking up.

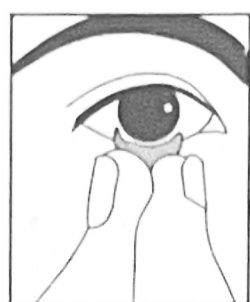

• The lens can then be pinched off the eye using the thumb and forefinger.

PRIOR TO LENS INSERTION

Wash and dry your hands before removing the lens from its container. Do not allow tap water to come into contact with the lens. Always remove contact lenses before swimming.

ATTENTION: Personal hygiene is very important.

DISINFECTION

Do not sleep with your lenses on. The lens is intended to be **WORN ONCE** and then **DISCARDED** at the end of each wearing period. You should start the next wearing period with a new lens. Reusing daily disposable contact lenses can increase the risk of infection, dryness and discomfort.

WARNING AND PRECAUTIONS

Remember regular check-up and always follow the advice of your Licensed Eye Care Practitioner.

Do not lend your lenses to anyone; they are only for one patient for a time.

Follow the wearing and replacement schedule as prescribed by your Licensed Eye Care Practitioner.

IF YOU EXPERIENCE EYE DISCOMFORT, EXCESSIVE TEARING, VISION CHANGES, OR REDNESS OF THE EYE, IMMEDIATELY REMOVE YOUR LENS AND PROMPTLY CONTACT YOUR LICENSED EYE CARE PRACTITIONER.

These instructions do not give you all the information you need for wearing contact lenses. It is very important to consult with your Licensed Eye Care Practitioner, and follow the prescription.

REPORTING OF ADVERSE REACTIONS

All serious adverse experiences and adverse reactions observed in patients wearing this contact lens or experienced with the lens should be reported to the manufacturer and the competent authority of the member state.

Clearlab SG Pte. Ltd.

139 Joo Seng Road, Singapore 368362.

Email: Regulatory@clearlab.com

IMPORTANT: Please read carefully and keep this information for future use.

Have your eyes examined regularly and follow your Eye Care Practitioner instruction for the proper use and care of your lenses. Please read the **PACKAGE INSERT** for important safety information **before use**. Ask your **Licensed Eye Care Practitioner** for a copy of **PACKAGE INSERT** or visit www.clearlab.com Lens care on how to do, proper cleaning, wearing, maintenance and storage.

Rx Only CAUTION: Federal Law (USA) restricts this device to sale by or on the order of a **Licensed Eye Care Practitioner**.



Sterilized using steam or dry heat




Single sterile barrier system



Do not use if package is damaged and consult instructions for use



European conformity sign

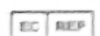

Authorized Representative in European Community



Consult instructions for use or consult electronic instructions for use



Manufacturer



Batch code (Lot number)



Do not re-use



Caution



Use by date (Expiry date)



Date of manufacture



Recyclable material



Temperature limits



Do not re-sterilize



Medical Device



Model number



Catalogue number



Unique device identifier



Keep away from sunlight



Keep dry



Fragile, handle with care



Diopter



Base Curve



Diameter